SUERO CONTROL DE BIOQUÍMICA I, II

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BIOCHEMISTRY CONTROL SERUM I, II

LEVEL I		LEVEL II	
COD 18005 20 x 5 mL	COD 18009 5 x 5 mL	COD 18007 20 x 5 mL	COD 18010 5 x 5 mL
Only for <i>in vitro</i> use in the clinical laboratory			

PRODUCT DESCRIPTION

The Biochemistry Control Serum is a lyophilized bovine serum containing component concentrations suitable for the quality control of the clinical laboratories, and without preservatives which might interfere with the tests. The product is intended for intralaboratory quality control purposes only and is supplied with intervals of suggested acceptable values.

CONTENTS AND COMPOSITION

Biochemistry Control Serum. Lyophilized bovine serum (for 5 mL).

INSTRUCTIONS FOR USE

1. Open a vial very carefully, avoiding any loss of the lyophilized material.
2. Pipette exactly 5.00 mL of distilled water into the vial. The component values depend on the accuracy of this reconstitution step.
3. Close the vial with the stopper and let stand for 20 minutes at room temperature.
4. Swirl gently, avoiding the formation of foam, to ensure complete dissolution of contents.
5. The reconstituted control serum is to be treated like the patient serum.

STORAGE AND STABILITY

Store at 2-8°C.

The lyophilized Biochemistry Control Serum is stable until the expiration date given in the label.

The components of the reconstituted Biochemistry Control Serum are stable for at least 7 days at 2-8°C, excepting:

– Alkaline phosphatase activity may increase with time.

Reconstituted Control Serum is stable for at least 30 days at -20°C. Freeze only once.

SPECIFIED VALUES

The assigned concentration values for components and their traceability are shown in the enclosed tables. Traceability can be assured only if the BioSystems reagents and recommended measurement procedures are used.

The intervals of suggested acceptable values have been calculated from previous experience in interlaboratory variability and are given for orientation only; each laboratory should establish its own precision parameters.

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