---

# CERTEST HAV

---

ONE STEP *Hepatitis A* virus  
CARD TEST

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## ENGLISH

### INTENDED USE

CerTest HAV one step card test is a coloured chromatographic immunoassay for the qualitative detection of *Hepatitis A* virus in stool samples. CerTest HAV card test offers a simple and a highly sensitive screening assay to make a presumptive diagnosis of *Hepatitis A* virus infection.

### INTRODUCTION

*Hepatitis A* is a liver infection caused by the *Hepatitis A* virus (HAV). The virus is spread when an uninfected (or unvaccinated) person eats or drinks something contaminated by the stool of an HAV-infected person: this is called faecal-oral transmission. The symptoms of *Hepatitis A* range from mild to severe and can include fever, malaise, loss of appetite, diarrhoea, nausea, abdominal discomfort, dark-coloured urine and jaundice (a yellowing of the skin and whites of the eyes). Not everyone who is infected will have all of the symptoms. Adults have signs and symptoms of illness more often than children and the severity of disease and mortality increases in older age groups. Infected children under six years of age do not usually experience noticeable symptoms, and only 10% develop jaundice. Among older children and adults, infection usually causes more severe symptoms, with jaundice occurring in more than 70% of cases. Most people recover in several weeks-or sometimes months-without complications.

### TEST PRINCIPLE

CerTest HAV is based on the principle of a qualitative immunochromatographic assay for the determination of *Hepatitis A* virus in stool samples. The strip consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against *Hepatitis A* virus and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-HAV) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming coloured conjugate complexes.

If the sample is positive, the antigens of the sample react with the red-coloured conjugate complex (anti-HAV monoclonal antibodies-red polystyrene microspheres), which was previously pre-dried on the absorbent pad. The mixture then moves upward on the membrane by capillary action. As the sample flows through the test membrane, the binding conjugate complexes migrate. The anti-HAV antibodies present on the membrane (test line) capture the coloured conjugate and the red line will be visible. This band is used to interpret the result.

If the sample is negative, there is no HAV antigens presence and yet, the antigens may be present in a concentration lower than the detection limit value, for which the reaction will not take place with the red-coloured conjugate complex. The anti-HAV antibodies present on the membrane (test line) will not capture the antigen-red-coloured conjugate complex (not formed), for which the red line will not appear.

Whether the sample is positive or not, the mixture continues to move across the membrane to the immobilized specific antibodies placed in the control line. The anti-specific protein antibodies present on the membrane will capture control green-conjugate complex and the control line will always appear. The presence of this green line serves as: 1) verification that sufficient volume is added, 2) that proper flow is obtained and 3) an internal control for the reagents.

### STORAGE AND STABILITY

Store as packaged in the sealed pouch at 2-30°C. The test is stable until the expiration date marked on its sealed pouch. The test must remain in the sealed pouch until use. Do not freeze.

### PRECAUTIONS

- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- All the specimens should be considered potentially hazardous and handled in the same manner as an infectious agent. A new test must be used for each sample to avoid contamination errors.
- The tests should be discarded in a proper biohazard container after testing.
- Reagents contain preservatives. Avoid any contact with skin or mucous membrane. Consult safety data sheet, available on request.
- Components provide in the kit are approved for use in the CerTest HAV test. Do not use any other commercial kit component.
- Follow Good Laboratory Practices, wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area.

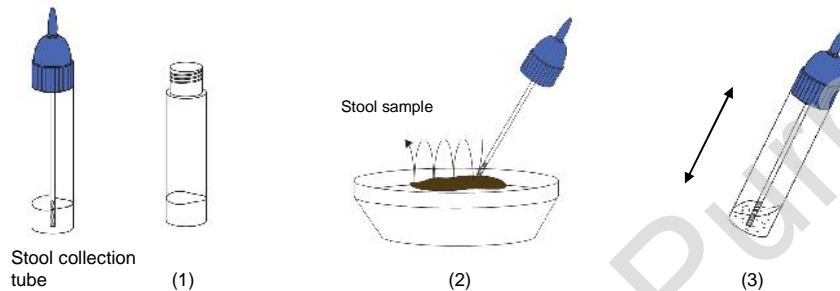
## SPECIMEN COLLECTION AND PREPARATION

Stool samples should be collected in clean containers. The samples can be stored in the refrigerator (2-8°C) for 1-2 days prior to testing. For longer storage, maximum 1 year, the specimen must be kept frozen at -20°C. In this case, the sample will be totally thawed and brought to room temperature before testing. Homogenise stool samples as thoroughly as possible prior to preparation.

### Specimen preparation (see illustration):

1. Take out the cap of the stool collection tube (1) and use the stick to pick up sufficient sample quantity. Then, introduce the stick once into 4 different parts of the stool sample (2), to collect faecal sample (approx. 125mg) and add it to the stool collection tube. For liquid samples, add approx. 125µL in the stool collection tube using a micropipette.
2. Close the tube with the diluent and stool sample. Shake the tube in order to assure good sample dispersion (3).

## MATERIALS

### MATERIALS PROVIDED

- CerTest HAV card tests
- Instructions for use
- Stool collection tubes with diluent

### MATERIALS REQUIRED BUT NO PROVIDED

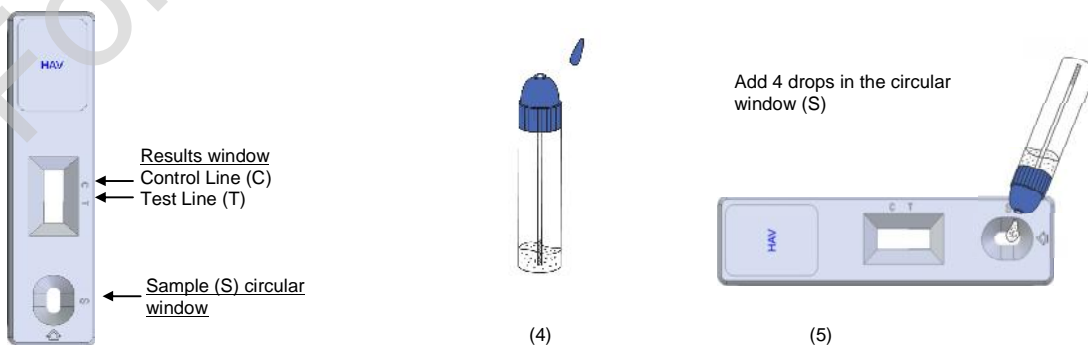
- Specimen collection container
- Disposable gloves
- Timer

## TEST PROCEDURE


Allow tests, stool samples and controls to reach room temperature (15-30°C) prior to testing. Do not open pouches until the performance of the assay.

1. Proceed to shake the stool collection tube in order to assure good sample dispersion.
2. Remove the CerTest HAV card test from its sealed bag just before using it.
3. Take the stool collection tube, cut the end of the cap (4) and dispense 4 drops in the circular window marked with the letter S (5). Avoid adding solid particles with the liquid.
4. **Read the results at 15 minutes.** Do not read the result later than 15 minutes.

If the test does not run due to solid particles, stir the sample added in the sample window (S) with the stick. If it doesn't work, dispense a drop of diluent until seeing the liquid running through the reaction zone.

## INTERPRETATION OF RESULTS (please refer to the illustration below)

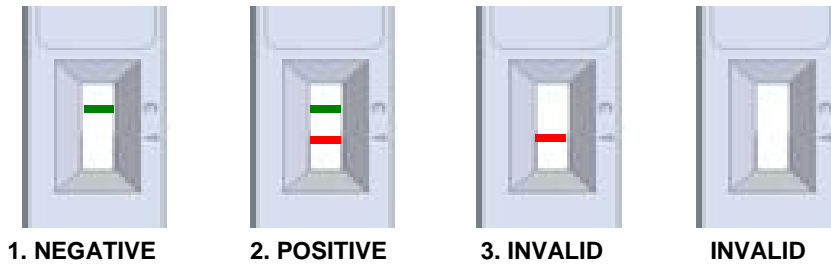	HAV	Interpretation of the results
1.	- GREEN	There is no HAV presence. No viral infection caused by HAV.
2.	+ GREEN-RED	
3.	Any other result	Invalid result, we recommend repeating the assay using the same sample with another test.

**NEGATIVE:** Only the GREEN line appears across the results window in the site marked with the letter C (control line).

**POSITIVE:** In addition to the GREEN control line, a RED line also appears in the site marked with the letter T (test line) in the results window.

**INVALID:** Total absence of the control coloured line (GREEN) regardless the appearance or not of the test line (RED). Insufficient specimen volume, incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the assay with a new test. If the symptoms or situation still persist, discontinue using the test kit and contact your local distributor.

### NOTES ON THE INTERPRETATION OF RESULTS

The intensity of the red coloured band in the test line (T) in the results window will vary depending on the concentration of antigens present in the specimen. However, neither the quantitative value nor the rate of increase in antigens can be determined by this qualitative test.

### QUALITY CONTROL

Internal procedural control is included in the test. A green line appearing in the control line (C) in the results window is an internal control, which confirms sufficient specimen volume and correct procedural technique.

### LIMITATIONS

1. The test must be carried out within 2 hours after opening the sealed bag.
2. An excess of sample could cause wrong results (brown bands appear). Dilute the sample with the diluent and repeat the test.
3. The intensity of test line may vary from very strong at high antigens concentration to faint when the antigens concentration is close to the detection limit value of the test.
4. After one month of infection, the number of viruses in faeces is decreasing, making the sample less reactive. Stool samples should be collected previously to the onset of symptoms to stop the spread of viruses.
5. CerTest HAV should be used only with samples from human faeces. The use of other samples has not been established. The quality of the test depends on the quality of the sample; proper faecal specimens must be obtained.
6. Positive results determine the presence of *Hepatitis A* virus in faecal samples; nevertheless, a positive result should be followed up with additional laboratory techniques to confirm the results. A confirmed infection should only be made by a physician after all clinical and laboratory findings have been evaluated and must be based in the correlation of the results with further clinical observations.
7. A negative result is not meaningful because of it is possible the antigens concentration in the stool sample is lower than the detection limit value. If the symptoms or situation still persist, a *Hepatitis A* virus determination should be carried out with another technique.

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## EXPECTED VALUES

The severity of acute HAV infections is proportional to the age of the patient, with younger patients tending to have milder disease than the elderly. Indeed, overall mortality rates are only 0.1% in the general population as opposed to 1–2% in the elderly. *Hepatitis A* infections do not progress to chronic liver disease (defined as hepatitis persisting beyond 6 months). Rarely, when the acute hepatitis is severe, patients may develop cholestasis (manifest by pruritus) which can persist for as long as 1 year following the initial injury.

Negative results are expected in healthy infants and young children, also in healthy adults.

## PERFORMANCE CHARACTERISTICS

### Clinical sensitivity and specificity

An evaluation, with faecal samples, was performed comparing the results obtained by an immunochromatographic test (CerTest HAV, CerTest) and another commercial HAV immunoassay (HAV-Antigen EIA, Mediagnost®). The results were as follows:

		HAV-Antigen EIA		
		+	-	Total
IC test: CerTest HAV	+	2	0	2
	-	0	24	24
	Total	2	24	26

	Sensitivity	Specificity	PPV	NPV
CerTest HAV vs HAV-Antigen EIA	>99%	>99%	>99%	>99%

The results showed a high sensitivity and specificity to detect HAV using CerTest HAV.

### Cross reactivity

An evaluation was performed to determine the cross reactivity of CerTest HAV; no cross reactivity against gastrointestinal pathogens occasionally present in faeces:

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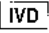
1. COULEPIS, A. et al. "Detection of *Hepatitis A Virus* and Antibody by Solid-Phase Radioimmunoassay and Enzyme-Linked Immunosorbent Assay with monoclonal antibodies". *Journal of Clinical Microbiology*, July 1985, Vol. 22, No. 1, p. 119-124.
2. BOROVEC, S. et al. "Synthesis and Assembly of *Hepatitis A Virus*-Specific Proteins in BS-C-1 Cells". *Journal of Virology*. June 1993, Vol. 67, No.6, p. 3095-3102.
3. G. Y. MINUK, A. J. COHEN, N. ASSY & M. MOSER. "Viral hepatitis and the surgeon". *Liver Diseases Unit, Departments of Medicine, Surgery and Pharmacology, University of Manitoba, Winnipeg, Manitoba, Canada*, 2005, p. 56-64.

## SYMBOLS FOR IVD COMPONENTS AND REAGENTS/SÍMBOLOS PARA REACTIVOS Y PRODUCTOS PARA DIAGNÓSTICO IN VITRO

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