CERTEST

Shigella

ONE STEP *Shigella* CARD TEST

CERTEST BIOTEC S.L.

For information purposes only

For information purposes only

CERTEST Shigella

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ENGLISH

INTENDED USE

CerTest *Shigella* one step card test is a coloured chromatographic immunoassay for the qualitative detection of *Shigella* (*S. dysenteriae*, *S. flexneri*, *S. boydi*, *S. sonnei*) in stool samples and *Shigella* suspected colonies in stool culture.

CerTest *Shigella* card test offers a simple and highly sensitive screening assay to make a presumptive diagnosis of *Shigella* infection (shigellosis) and it could be used to identify of suspected isolates of *Shigella* from selective media (stool culture).

INTRODUCTION

The four species of the genus *Shigella*; *S. dysenteriae*, *S. flexneri*, *S. boydii* and *S. sonnei* cause a wide spectrum of illness from watery diarrhoea to fulminant dysentery. The low infectious inoculum, as few as 10 organisms, render *Shigella* highly contagious. Shigellosis therefore occurs as an endemic disease in populations characterized by over-crowding, poor housing, poor sanitation and inadequate water supply. The predominant serogroups of *Shigella* occurring in a region also appears to be related to the socioeconomic development; and evidence also indicates that the severity of shigellosis is related to the infecting serogroup.

For example, *S. dysenteriae* type 1, also known as Shiga bacillus, has been recognized as the major cause of epidemic dysentery for nearly 100 years. Pandemics of Shiga dysentery have spread across Central America, Bangladesh, South Asia and Central and East Africa.

TEST PRINCIPLE

CerTest *Shigella* is based on the principle of a qualitative immunochromatographic assay for the determination of *Shigella* in stool samples and *Shigella* suspected colonies in stool culture.

The strip consists of a nitrocellulose membrane pre-coated with antibodies on the test line (T), in the results window, against *Shigella* and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (antibodies anti-*Shigella*) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming coloured conjugate complexes.

If the sample is positive, the antigens of the diluted sample react with the red-coloured conjugate complex (anti-*Shigella* antibodies-red polystyrene microspheres) which was previously pre-dried on the absorbent pad. The mixture then moves upward on the membrane by capillary action. As the sample flows through the test membrane, the binding conjugate complexes migrate. The anti-*Shigella* antibodies present on the membrane (test line) capture the coloured conjugate and the red line will be visible. This band is used to interpret the result.

If the sample is negative, there is no *Shigella* antigens presence and yet, the antigens may be present in a concentration lower than the detection limit value, for which the reaction will not take place with the red-coloured conjugate complex. The anti-*Shigella* antibodies present on the membrane (test line) will not capture the antigen-red-coloured conjugate complex (not formed), for which the red line will not appear.

Whether the sample is positive or not, the mixture continues to move across the membrane to the immobilized specific antibodies placed in the control line. The anti-specific protein antibodies present on the membrane will capture control green-conjugate complex and the control line will always appears. The presence of this green line serves as: 1) verification that sufficient volume is added, 2) that proper flow is obtained and 3) an internal control for the reagents.

STORAGE AND STABILITY

Store as packaged in the sealed pouch at 2-30°C. The test is stable until the expiration date marked on its sealed pouch. The test must remain in the sealed pouch until use. Do not freeze.

PRECAUTIONS

- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- All the specimens should be considered potentially hazardous and handled in the same manner as an infectious agent. A new test must be used for each sample to avoid contamination errors.
- The tests should be discarded in a proper biohazard container after testing.
- Reagents contain preservatives. Avoid any contact with skin or mucous membrane. Consult safety data sheet, available on request.
- Components provide in the kit are approved for use in the CerTest *Shigella* test. Do not use any other commercial kit component.
- Follow Good Laboratory Practices, wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area.


SPECIMEN COLLECTION AND PREPARATION

Stool samples

Samples should be collected in clean containers. The samples can be stored in the refrigerator (2-8°C) for 1-2 days prior to testing. For longer storage, maximum 1 year, the specimen must be kept frozen at -20°C. In this case, the sample will be totally thawed and brought to room temperature before testing. Ensure only the amount needed is thawed because of freezing and defrosting cycles are not recommended. Homogenise stool samples as thoroughly as possible prior to preparation.

Specimen preparation (see illustration):


1. Take out the cap of the collection tube (1) and use the stick to pick up sufficient sample quantity. Then, introduce the stick once into 4 different parts of the stool sample (2), to collect faecal sample (approx. 125mg) and add it to the collection tube. For liquid samples, add approx. 125µL in the collection tube using a micropipette.
2. Close the tube with the diluent and stool sample. Shake the tube in order to assure good sample dispersion (3).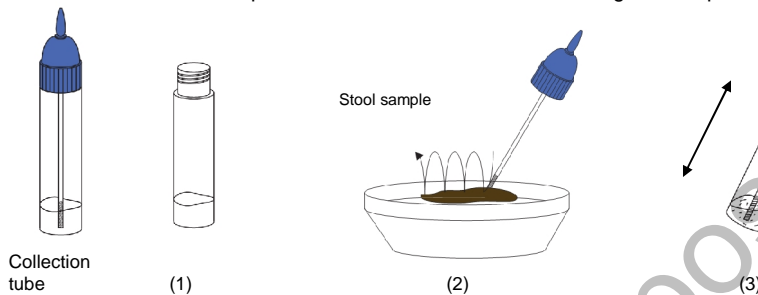


Suspected *Shigella* colonies in stool culture

The Hektoen Enteric Agar (HEA) is the method of choice for the isolation of *Shigella* (aerobic atmosphere, 24 hours/37°C). After 24 hours of incubation in Hektoen the typical *Shigella* colonies will be green or bluish without any black centre.


1. Examine Hektoen plates after 1 day incubation. Select *Shigella* typical colonies. Take out the cap of the collection tube (1). Use the stick or an inoculating needle to pick up 3 or 4 suspected *Shigella* colonies (2) and add them to the collection tube.
2. Close the tube with the diluent and suspected colonies. Shake the tube in order to assure good sample dispersion (3).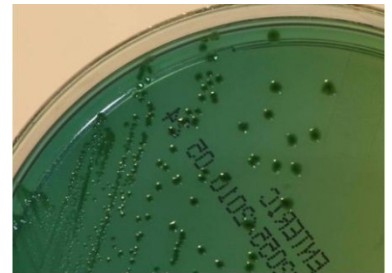

Shigella colonies in Hektoen Enteric Agar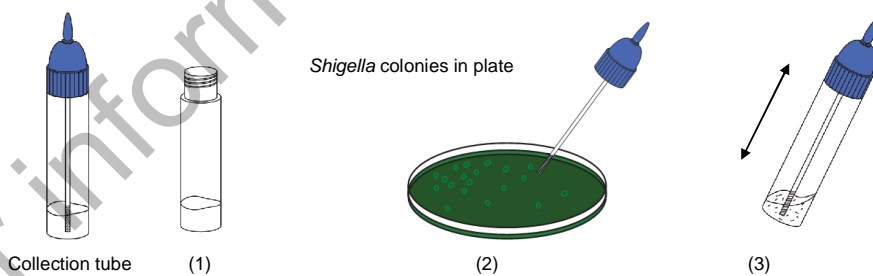


Salmonella Shigella Agar (SS) is another method for the isolation of *Shigella* (aerobic atmosphere, 24-48 hours/35-37°C). After the incubation period in SS the typical *Shigella* colonies will be transparent without any black center. The SS agar is a very inhibitor that sometimes prevents the growth of certain species of *Shigella*. We recommend using this medium unless accompanied by another inhibitor (HEA).

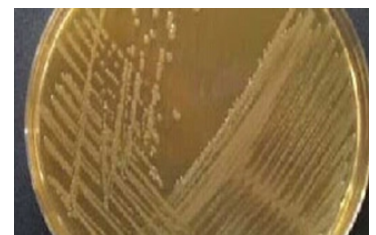Shigella colonies in *Salmonella Shigella* Agar

1. Examine SS plates after incubation period. Select *Shigella* typical colonies. Take out the cap of the collection tube (1). Use the stick or an inoculating needle to pick up 3 or 4 suspected *Shigella* colonies (2) and add them to the collection tube.
2. Close the tube with the diluent and suspected colonies. Shake the tube in order to assure good sample dispersion (3).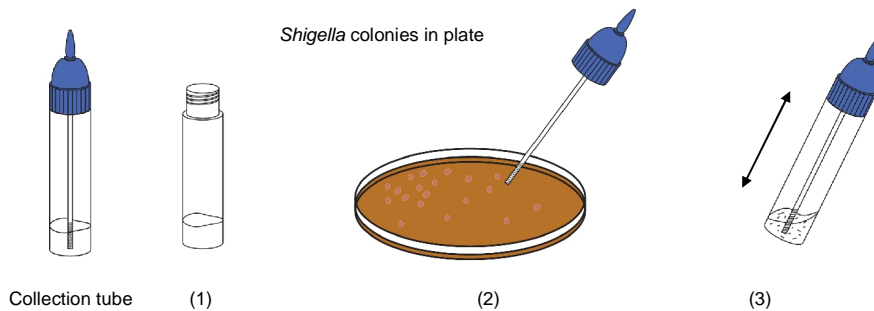

MATERIALS

MATERIALS PROVIDED

- CerTest *Shigella* card tests
- Instructions for use
- Collection tubes with diluent

MATERIALS REQUIRED BUT NO PROVIDED


- Specimen collection container
- Disposable gloves
- Timer
- Hektoen Enteric Agar (HEA)/Salmonella Shigella Agar (SS)

TEST PROCEDURE


Allow tests, samples and controls to reach room temperature (15-30°C) prior to testing. Do not open pouches until the performance of the assay.

1. Proceed to shake the collection tube in order to assure good sample dispersion.
2. Remove CerTest *Shigella* card test from its sealed bag just before using it.
3. Take the collection tube, cut the end of the cap (4) and dispense 4 drops in the circular window marked with the letter S (5). Avoid adding solid particles with the liquid.
4. **Read the results at 10 minutes.** Do not read the test result later than 10 minutes.

If the test does not run due to solid particles, stir the sample added in the sample window (S) with the stick. If it doesn't work, dispense a drop of diluent until seeing the liquid running through the reaction zone.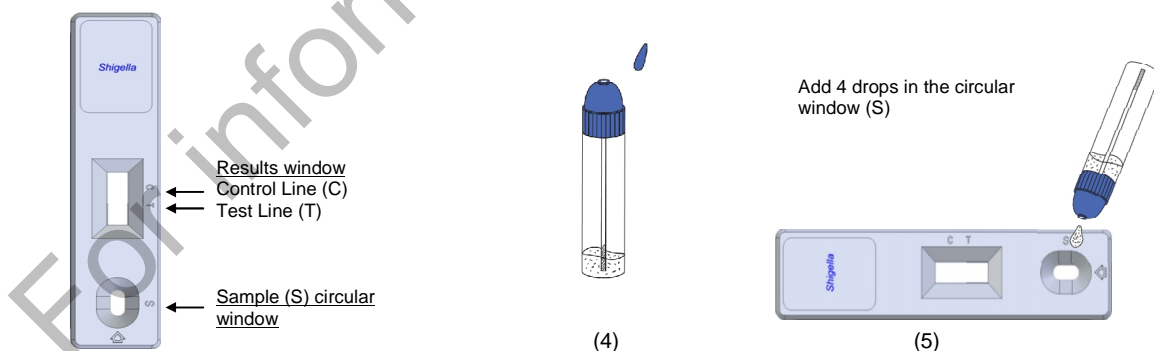

INTERPRETATION OF RESULTS (please refer to the illustration below)

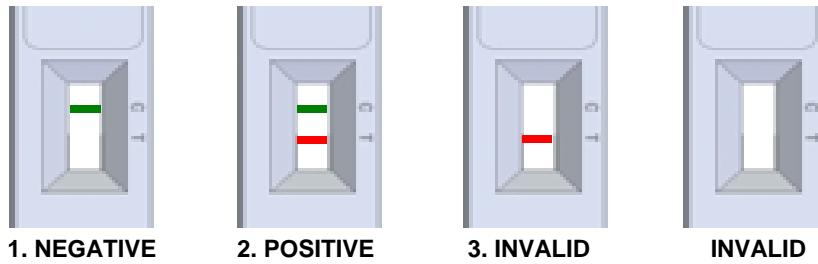	<i>Shigella</i>	Interpretation of the results
1.	- GREEN	There is no <i>Shigella</i> presence. No infection caused by <i>Shigella</i> .
2.	+ GREEN-RED	There is <i>Shigella</i> presence. <i>Shigella</i> infection which might include from watery diarrhoea to fulminant dysentery.
3.	Any other result	Invalid result, we recommend repeating the assay using the same sample with another test.

NEGATIVE: Only the GREEN line appears across the results window in the site marked with the letter C (control line).

POSITIVE: In addition to the GREEN control line, a RED line also appears in the site marked with the letter T (test line) in the results window.

INVALID: Total absence of the control coloured line (GREEN) regardless the appearance or not of the test line (RED). Insufficient specimen volume, incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the assay with a new test. If the symptoms or situation still persists, discontinue using the test kit and contact your local distributor.

NOTES ON THE INTERPRETATION OF RESULTS

The intensity of the red coloured band in the test line (T) in the results window will vary depending on the concentration of antigens present in the specimen. However, neither the quantitative value nor the rate of increase in antigens can be determined by this qualitative test.

QUALITY CONTROL

Internal procedural control is included in the test. A green line appearing in the control line (C) in the results window is an internal control, which confirms sufficient specimen volume and correct procedural technique.

LIMITATIONS

- The test must be carried out within 2 hours after opening the sealed bag.
- An excess of sample could cause wrong results (brown bands appear). Dilute the sample with the buffer and repeat the test.
- The intensity of test line may vary from very strong at high antigens concentration to faint when the antigens concentration is close to the detection limit value of the test.
- CerTest *Shigella* should be used only with samples from human faeces. The use of other samples has not been established. The quality of the test depends on the quality of the sample; proper faecal specimens must be obtained.
- Positive results determine the presence of *Shigella* in faecal samples; nevertheless, it can be due to several causes and/or species. A positive result should be followed up with additional laboratory techniques (biochemical and serological methods or by PCR) to confirm the results. A confirmed infection should only be made by a physician after all clinical and laboratory findings have been evaluated and must be based in the correlation of the results with further clinical observations.
- A negative result is not meaningful because of it is possible the antigens concentration in the stool samples is lower than the detection limit value. If the symptoms or situation still persist, a *Shigella* determination should be carried out on a sample from an enrichment culture.

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EXPECTED VALUES

Bacillary dysentery caused by *Shigella dysenteriae* type 1 is a major public health problem in many developing countries. There are four known species of *Shigella* which are pathogenic, and infection with *S. dysenteriae* type 1 usually progresses to the most severe form of dysentery with life-threatening complications. The most common underlying cause of death in fatal shigellosis is severe colitis and the immediate associated causes are septicemia and pneumonia. Infection with *S. dysenteriae* type 1 can occur in an epidemic form, and *Shigella*-contaminated food and drink are often the source of epidemic spread.

PERFORMANCE CHARACTERISTICS

Analytical sensitivity (detection limit)

Detection limit value for the different serotypes is: 3.125×10^4 CFU/mL for *S. dysenteriae*, 7.8×10^5 CFU/mL for *S. Flexneri*, 2.5×10^9 CFU/mL for *S. sonnei* and 1×10^{10} CFU/mL for *S. boydii*.

Clinical sensitivity and specificity

An evaluation, with faecal samples, was performed using bacteria cultures *in vitro* to detect *Shigella* infection. The results were as follows:

IC test: CerTest <i>Shigella</i>	Culture confirmation		Total
	+	-	
+	5	0	5
-	0	10	10
Total	5	10	15

	Sensitivity	Specificity	PPV	NPV
CerTest <i>Shigella</i> vs Culture	>99%	>99%	>99%	>99%

The results showed a high sensitivity and specificity to detect *Shigella* using CerTest *Shigella*.

Cross reactivity

An evaluation was performed to determine the cross reactivity of CerTest *Shigella*; no cross reactivity against gastrointestinal pathogens occasionally present in faeces:

<i>Escherichia coli</i> O157:H7	<i>Salmonella typhi</i>	<i>Salmonella paratyphi</i>
<i>Helicobacter pylori</i>	<i>Salmonella typhimurium</i>	<i>Salmonella enteritidis</i>
<i>Listeria monocytogenes</i>	<i>Staphylococcus aureus</i>	<i>Yersinia enterocolitica</i>

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