

# Val-A-Sure™ Cooler Validation Kit

Accurate. Consistent. Simple.




## QuickStart Guide



**STOP** Have you configured the software? See other side for Steps 1 - 3 before completing these steps.


### Cooler Validation Instructions with Optional Probes

#### 4 Assemble Equipment & Supplies

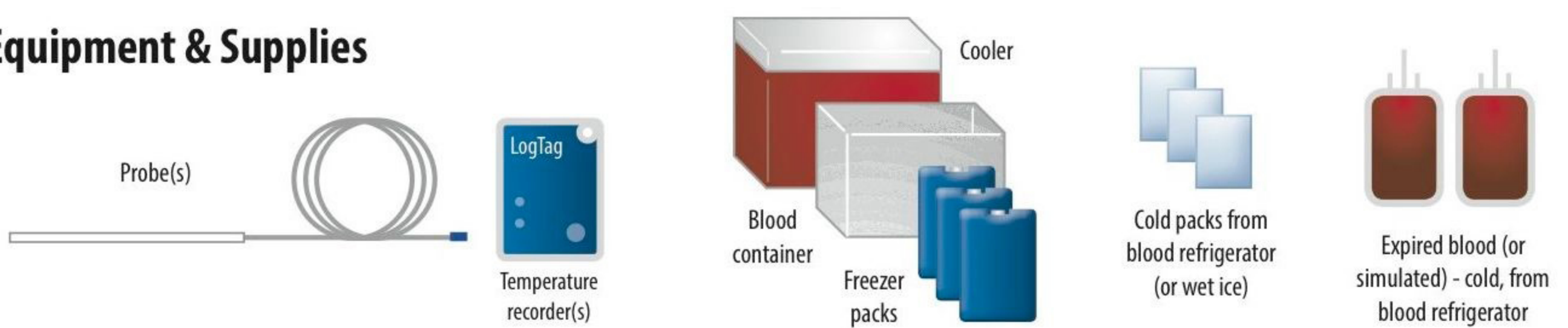

#### 5 PACK COOLER and START Temperature Recording

- Place container (for blood units) in cooler and surround with freezer packs
- Insert probe in topmost bag, attach probe to recorder
- Press START button on the recorder. Flashing red then green "OK" indicates temperature is recording.
- Pack cooler with blood units, recorder and cold packs as shown.

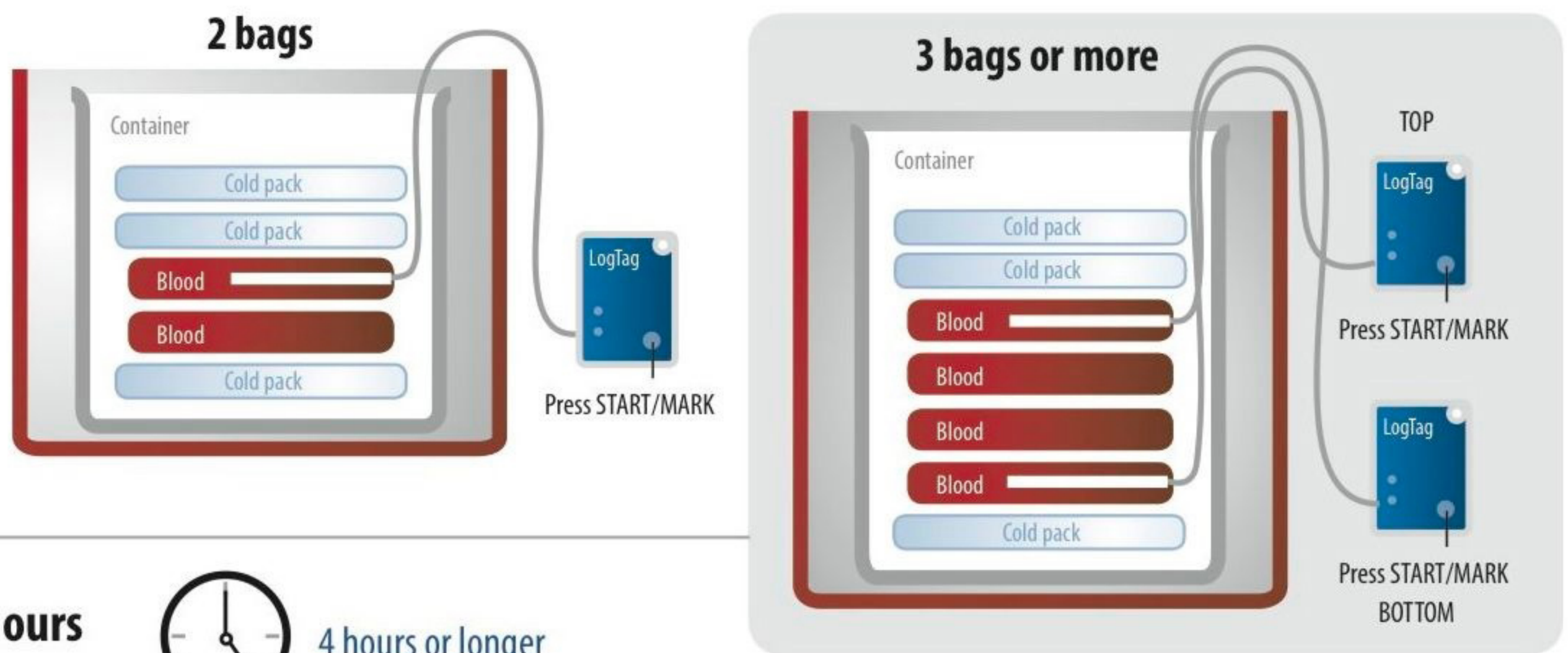

#### 6 CLOSE COOLER and wait for 4 hours (or longer depending on your validation protocol)



#### 7 COLLECT recorder and DOWNLOAD Temperature Data to computer



- Launch software on your computer
- Remove recorder from cooler and place in cradle
- Data will automatically download and display

#### 8 VIEW and PRINT Temperature Data, attach to Validation Documents




#### 9 Hibernate Recorder

To extend battery life

While Recorder is still in cradle:

- Click on **LogTag** in top menu
- Scroll down to select **"Hibernate"**

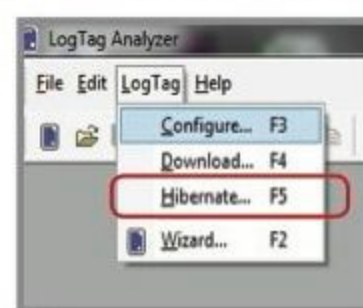

for more information: [www.williamlabs.com](http://www.williamlabs.com)